



SOTOXA ORIENTATION, TRAINING, SETUP

Intoximeters
— EXPERIENCE SERVICE INTEGRITY —



SOTOXA™ MOBILE TEST SYSTEM INITIAL SETUP

SOTOXA™ MOBILE TEST SYSTEM OVERVIEW




QUALITY CONTROL OR QC TESTING

QC cartridges act as a tool to confirm:

Camera in the
analyzer is aligned.

Internal lighting is
correct.

Analyzer is able to
correctly interpret
the test results.

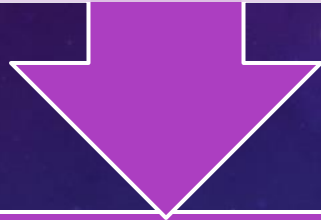

It is recommended that QC testing is conducted at least once on each day of testing using the NQC and PQC cartridges.



ONLINE USER TRAINING

Online User Training is available from Abbott

REGISTRATION LINK:

Use the link below to create a **new** login.

The password is **welcome**

<http://learningxchange.aleretoxicology.com/group/login/roadside>

You may utilize this link after registering with your username and password.



SOTOXA™ MOBILE TEST SYSTEM



Cookie Preferences

ROADSIDE SOLUTIONS WELCOME TO LEARNING XCHANGE



SIGN UP

Already have an account? [Sign in](#)

1st Time Login



Cookie Preferences

© 2020 Abbott. All rights reserved. The Abbott logo, Abbott and Learning XChange are trademarks of the Abbott group of companies. Other trademarks are the property of their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. Your use of this website and the information contained herein is subject to our [Website Terms and Conditions](#), [Privacy Policy](#) and [Consumer Health Data Privacy Policy](#).

ROADSIDE SOLUTIONS WELCOME TO LEARNING XCHANGE



SIGN UP

Already have an account? [Sign in](#)

Click Sign in after first login and
account creation

© 2020 Abbott. All rights reserved. The Abbott logo, Abbott and Learning XChange are trademarks of the Abbott group of companies. Other trademarks are the property of their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. Your use of this website and the information contained herein is subject to our [Website Terms and Conditions](#), [Privacy Policy](#) and [Consumer Health Data Privacy Policy](#).

ONLINE USER TRAINING

Upon completion of training, a certificate will be generated and sent to your registered e-mail address as a PDF.



Abbott

CERTIFICATE OF COMPLETION

This certifies that

Rob Duckworth

Scored 12 / 12 and Satisfactorily Completed
SoToxa¹™ Oral Fluid Mobile Test System

on 12/29/2023

Training completed on the Learning XChange™ training portal.

SOTOXA™ MOBILE TEST SYSTEM

Most Important for Your Success with SoToxa:

Visibility of blue on the collector must
be present for a successful test.

*At a minimum, a portion of the
collector must be visibly blue.

Intoximeters
— EXPERIENCE SERVICE INTEGRITY —

If you do not see **BLUE**, it
will not **DO**!



Need assistance or have questions?

Contact our team



[HTTPS://WWW.INTOX.COM/PRODUCT/SOTOX
A-MOBILE-TEST-SYSTEM/](https://www.intox.com/product/sotox-a-mobile-test-system/)

Regional Managers

East Region (States: ME, VT, NH, MA, NY, CT, NJ, RI, DE, MD, DC, VA, PA)

Jim McCauley, Regional Manager

(315) 607-3279

jmccauley@intox.com

Southeast Region (States: TN, MS, AL, SC, NC, GA, FL, AR, LA)

Byron Greenwood, Regional Manager

(256) 419-3593

bgreenwood@intox.com

Upper Midwest Region (States: MI, IL, IN, OH, KY, WV)

Rob Duckworth, Regional Manager

(812) 449-8823

rduckworth@intox.com

Midwest Region (States: MO, KS, IA, NE, MN, WI, OK)

Tim Brewer, Regional Manager

(314) 409-7901

tbrewer@intox.com

Mountain Region (States: MT, WY, UT, CO, NM, ID, ND, SD, TX)

Mark Postal, Regional Manager

(314) 406-2864

mpostal@intox.com

West Region (States: CA, NV, AZ, OR, WA, AK, HI)

Todd Hall, Regional Manager

(909) 556-6682

thall@intox.com